

BIOMECHANICAL SOLUTIONS

Not recommended for this device

Diagnosis	Heel Cup	Width	Medical Arch Height	Heel Skive	Rearfoot Post	Forefoot Post	Heel Lift	Length	Modifications
Achilles Tendinitis	14mm	Medium	Lower 1/8	4mm	0/0	Intrinsic	1/8	Mets	
Calcaneal Apophysitis	18mm	Wide	Tight to Cast		0/0	Intrinsic	1/8	Mets	Memocell heel pad
Hallux Limitus / HAV	N/A	Narrow	Tight to Cast		N/A	Intrinsic		Sulcus	1 st Met cut out, rev. Morton's ext.
Intoeing Gait (Gait plate)	14mm	Wide	Lower 1/8		0/0	Intrinsic		Mets	Gait ext. cause toe out
Lateral Ankle Instability / Peroneal Tendinitis	14mm	Wide	Lower 1/8		0/0	3° Valgus corner post		Mets	Lateral flange
Metatarsalgia	N/A	Narrow	Tight to Cast		N/A	Intrinsic		Sulcus	Metatarsal pad
Neuroma	N/A	Narrow	Tight to Cast		N/A	Intrinsic		Sulcus	Neuroma pad (3 rd interspace) short 1 st mpj add Morton's ext.
Pediatric Flatfoot	18mm	Wide	Tight to Cast	4mm	4/4	Intrinsic		Mets	Medial flange
Pes Cavus / Rigid Forefoot	N/A	Narrow	Lower 1/8		N/A	3° Valgus corner post	N/A	Sulcus	Metatarsal pad
Fasciitis / Everted Rearfoot	18mm	Medium	Tight to Cast	4mm	4/4	Intrinsic		Mets	
Fasciitis / Forefoot Valgus	N/A	Narrow	Lower 1/8		N/A	3° Valgus corner post		Sulcus	1 st Met cut out
Posterior Tibialis Dysfunction	18mm	Wide	Lower 1/9	4mm	4/4	4° Valgus corner post		Mets	Medial flange
Sesamoiditis	N/A	Narrow	Tight to Cast		N/A	Intrinsic		Sulcus	1 st Met cut out, rev. Morton's ext.
Sinus Tarsi Syndrome	18mm	Medium	Tight to Cast	4mm	4/4	Intrinsic		Mets	
Tarsal Tunnel Syndrome	18mm	Medium	Tight to Cast	4mm	2/2	Intrinsic	1/8	Mets	

800-732-5446

America's Value Leader In Custom Orthotics

